## Wichtige Informationen/Garantie

### Vorsicht ⚠

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## Important information/Guarantee

### Attention ⚠

- Subtilis chisel blades are extra-hard and finely ground. To avoid the risk of breaking a blade, do not apply leverage.

- accuratus and Subtilis instruments with self-holding /fixation mechanism (exemple images),

have a limited time of use for 60 minutes at most during each invasive application.

### Guarantee ⓘ

All Subtilis instruments are manufactured with the utmost care and they satisfy the strictest quality criteria of Accuratus AG. The maker grants a five year guarantee covering demonstrable defects in material or manufacture. The guarantee will not apply to any claims relating to the instruments which arise from improper use or handling, modifications or defective third party repairs.

### Cleaning and care ⓘ

The instruments are delivered in a **NON-STERILE** condition. They must be cleaned and sterilised before use. Sharp points, cutting edges ect, must be covered with protective caps or tubes as appropriate. Sterilise and store the instruments with the lock open. After every cleaning, the instrument closure must be lubricated using an acid-free oil. Our general terms and conditions and important information on the care and handling of the instruments may be found on [www.accuratus.ch](http://www.accuratus.ch) or [www.subtilis.com](http://www.subtilis.com)

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made by Accuratus AG
Manufactured by: **Accuratus AG**